


1. Remove the blue upper section of the cap to open the China-Oel Inhalator.
2. Pull out the upper section of the cap together with the absorbent fleece.

3. Distribute 2–3 drops of China-Oel along the fleece tube.
4. Replace the upper section of the cap together with the absorbent fleece in the lower section of the inhalator and close it.



5. Treatment of the nasal cavity: Insert the **blue side** into each nostril and breathe through the **nose** vigorously.

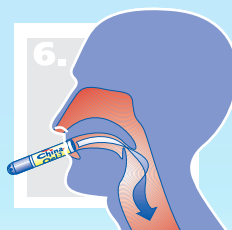

6. Treatment of the throat: Insert the **white side** of the inhalator into the **mouth** and breathe **deeply**.

Interesting information about the China-Oel Inhalator

The China-Oel Inhalator is used for inhalation of China-Oel. Inhalation brings the China-Oel into contact with the nasal and oral mucosa and deep inhalation introduces it into the upper respiratory tract, where it can then unfold its pharmacological effect.

Care and cleaning of the China-Oel Inhalator:

After using the China-Oel Inhalator, wipe off the outside of the device with a clean cloth and store it in the original package!

Travelling with the China-Oel Inhalator:

The advantage of the China-Oel Inhalator is that the application of the China-Oel is possible anywhere and anytime, e.g. at the office, in the theatre or car or while travelling. The inhalator is simple and easy to use and can be refilled with 2–3 drops of China-Oel.

Important:

Insert the inhalator only about 1 cm into the nasal openings or mouth. Do not swallow it! Store out of reach of children! Take precautions to make sure China-Oel does not get into the eyes when applying drops of it to the fleece tube!